

# Igenity<sup>®</sup> Testing Procedure

## Hair Collection Instructions

1. Pull (do not cut) hair from **tail switch**, not tail head.



2. Make sure hair is free of fecal material and dirt. Roots must be clearly visible. Approximately **20 to 30 hair roots** are needed.



3. Write the **animal's ID number** on the collector in the space provided.



4. Open the collector and **peel back the plastic cover**.



5. Insert the **root end** of the hair in the middle of the collector. Close the collector by sealing the plastic cover over the hair and then insert the tab into the slot.



6. **Trim the excess hair** extending from the collector. Put all collectors in the plastic specimen bag for shipment.



Please call for instructions on shipping tissue samples other than hair.

©Neogen and Igenity are registered trademarks of Neogen Corporation.  
©2012 Neogen Corporation. Lansing, MI. All rights reserved.

AG041-0812

[www.igenity.com](http://www.igenity.com)

1-877-IGENITY

 **Igenity**<sup>®</sup>  
a Neogen<sup>®</sup> Company